



Body Position Moving Mats M64RK Series

TEIJIN

Human Chemistry, Human Solutions

M64RK01 (Water Resistant)
Mini • Mini 52×52cm



Used on top of bed to correct body position

M64RK02 (Water Resistant)
Mini 80×62cm
(Folds in Half / Bag included)



Used on top of bed to move and change body position

M64RK04 (Water Resistant)
Mini • Long 120×40cm
(Folds in Half / Bag included)



Best for entering a bath – board material in center used

M64RK05 (Water Resistant)
Regular 180×55cm
(Fold into Thirds / Bag included)



For moving from a bed to a stretcher

M64RK07 (Water Resistant)
Large 80×72cm
(Fold into Thirds / Bag included)



For moving from a bed to a stretcher

M64RK11 (Water Resistant)
Type • 1 33×58cm




For moving from a bed to a wheelchair

M64RK12 (Water Resistant)
Mini • Seat 33×42cm



For moving while seated

M64RK14 (Water Resistant)
Slide Unit 180×40cm
(Fold into Thirds / Bag included)

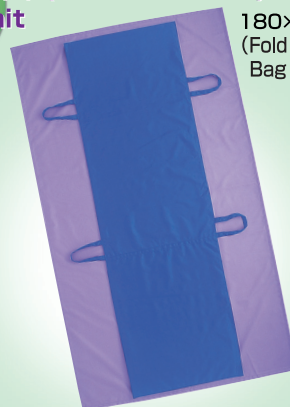

Slide System for moving longer distances

Move with ease and comfort without any lifting



Body Position Moving Mats M64RK Series

MADE IN JAPAN

●Material supplied by
TEIJIN FRONTIER CO., LTD.

●Manufacturer
TEIKEN LIMITED

Nakanoshima Festival Tower West
2-4, Nakanoshima 3-chome, Kita-Ku, Osaka 530-8605, Japan
TEL +81-6-6233-3565 FAX +81-6-6233-5108

Corporate Member of the Japanese No Lifting Association

●For Sales inquires
CMT Japan. Co., Ltd.

info@cmtco.jp
<http://www.cmtco.jp>

Body Position Moving Mats
M64RK Series Rolling System


● Comfortably move while lying down without any lifting.
These sliding mats are to help support caregivers so that no lifting of the body is needed. Teijin is able to develop these products by combining two easy to slide materials. Lifting bodies not only put stress on caregivers but for those being taken care of, it can be dangerous as well. M64RK SLIDING UNIT is a must-have product to provide a safe and comfortable caregiving environment.



● Easy to use; moving body positions is smooth.
A rolling sheet that turns around a center board makes moving body positions easy with ease. Various types are available, not just for use on beds only, but for moving from a bed to a stretcher, or a wheelchair, depending on the situation.





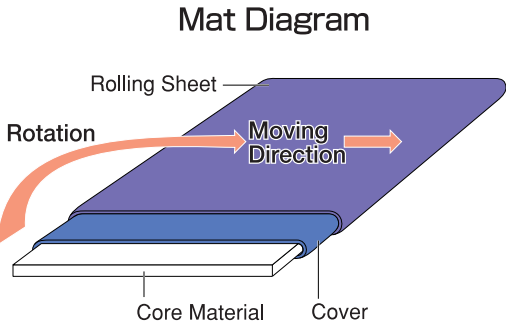




























- Characteristics of M64RK series
- A center board material, Teijin's high function cushion is also used, which helps maintain a soft contact between the body and mat.(Excludes M64RK11 and M64RK12)
 - With a center board material inside the mat, even if the surface is uneven, moving the mat is still smooth.(Excludes M64RK01, M64RK02, M64RK07)
 - Mats do not need to be removed when going thru an X-ray. (The center board should be taken out first).
 - Convenient bags are included when carrying the mats around or not in use. (Excludes M64RK01, M64RK11, M64RK12)



● Product Specification

● Product Specification are subject to change without prior notification.

Product Number	Product Name	Size (cm) L x W	Weight	Product Contents				Bag Included
				Sheet	Mat	Core Material		
						Elk	Board	
M64RK01	Mini・Mini (Water Resistant)	52× 52	475 g				—	—
M64RK02	Mini (Water Resistant)	80× 62	1,060 g				—	
M64RK04	Mini・Long (Water Resistant) <small>(Sheet replacement also sold separately)</small>	120× 40	1,960 g					
M64RK05	Regular (Water Resistant) <small>(Sheet and Board replacement also sold separately)</small>	180× 55	3,410 g					
M64RK07	Large (Water Resistant) <small>(Sheet replacement also sold separately)</small>	180× 72	2,400 g				—	
M64RK11	Type・1 (Water Resistant)	33× 58	1,120 g			—	 2 pcs	—
M64RK12	Mini・Seat (Water Resistant)	33× 42	460 g			—		—
M64RK14	Slide Unit (Water Resistant)	180× 40 184× 110	2,160 g	 2 sheets			—	

● Materials

Rolling Sheet		Polyester 100%	Uses Teijin's Anti-Static Material
Cover			
Bag			
Core Material	Cushion	Polyester 100%	Uses Teijin's High Function Cushion
	Board	Polypropylene 100%	PAULOWNIA (Foam molded polypropylene sheet, expanded 3 times)


※Product have been specially treated by Teijin Frontier to be stain resistant and durable in a washing machine. Please wash based on the laundry symbols on the right. When washing the cover (of the core material) be sure to remove the core material.




Usages




● Moving from a bed to a stretcher (M64RK07, M64RK05, M64RK04)

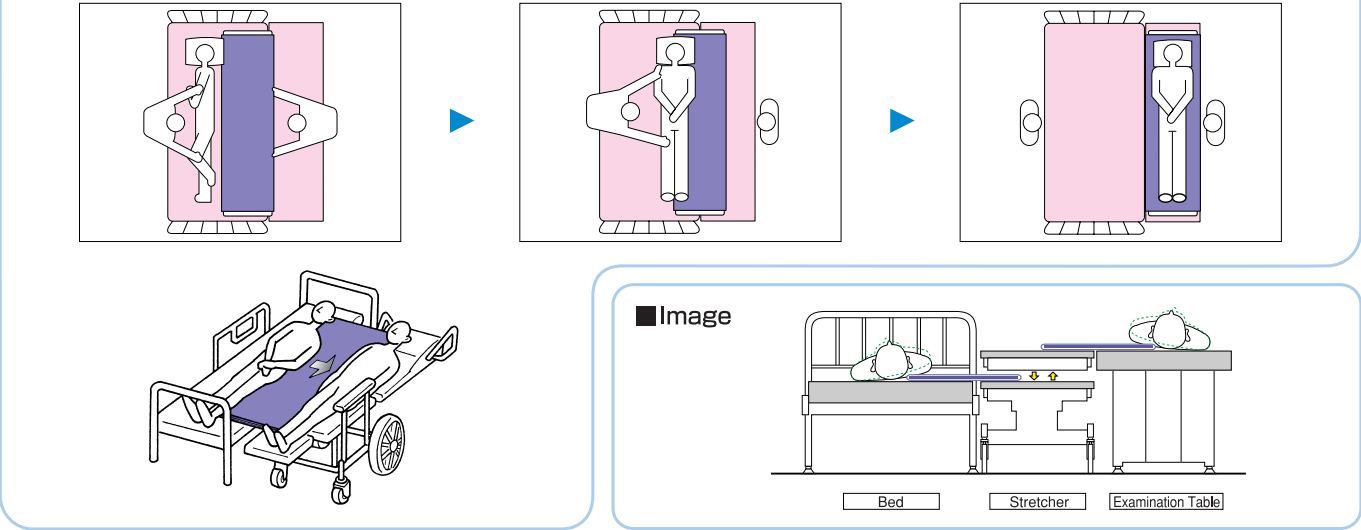


● Changing Body Position on a Bed (M64RK01, M64RK02) ● Using a Slide Unit (M64RK14)

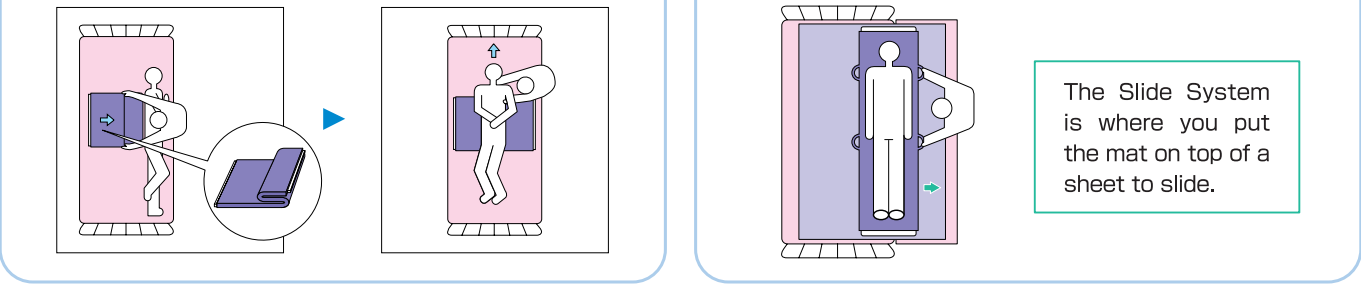

● Transferring to a Car (M64RK11)

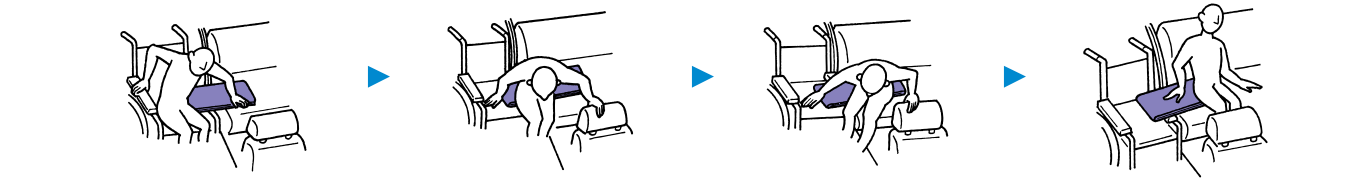

● Entering into a Bath (M64RK) ● Moving to a Wheelchair (M64RK11)

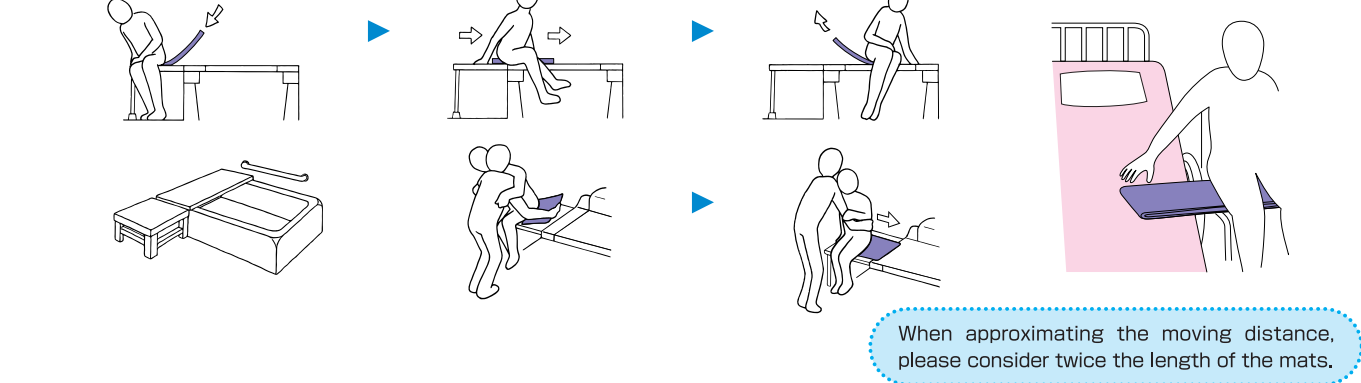

In Order to Use the Products Safely

- Warning** Improper use of the products may result in death or severe injuries due to possibility of an accident of falling off or toppling over.
- Do not use apart from using it to move or assisting someone else. Also, please remove the product immediately after use.
 - Do not leave the product laying on the stretcher, bed, or examination table when not in use.
 - Please check to see if the wheels are locked on the bed and stretcher, etc., before using the products.
 - When moving from a bed to a stretcher, make sure to limit the space between the two as much as possible.
 - M64RK14 Body Position Moving Mat Slide Unit is used to assist someone on the mat to slide. It should not be used for someone to lay on and be carried as a stretcher.
 - Make sure to be either sitting up straight or lying down when the products are being used.
 - Please makes sure that the products do not fall into the hands of young children.
- Caution** Improper use of the products may cause injury or damage to product itself.
- Do not use the mat or sheets separately. Also, do not insert anything other than the specified core material into the mat.
 - Do not insert anything in between the rolling sheet and mat.
 - When not in use, please store it back into the bag that is provided.
 - Please not use close to a fire source as it may light on fire.
 - Please remove the core material when cleaning.
 - Please follow the laundry symbols when cleaning and store properly. when not in use. (To prevent infections and damage to the products.)